



# **GECKO TRAXX**

User Guide


# Fitting Instructions

1.




At the edge of the sand (or any soft surface) take Gecko Traxx out of the mesh bag provided and unravel.

2.

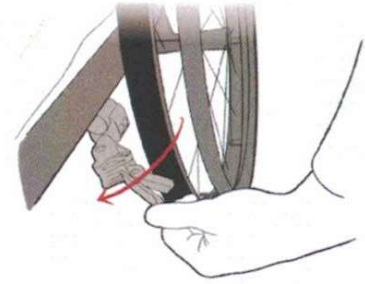

Make sure all obstructions (such as brakes) are out of the way of your tyre.

3.

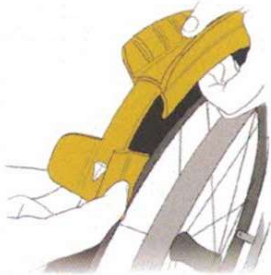

Take the end of Gecko Traxx with the small arrow facing forward and fit the clip around your tyre. Push down on the clip so that it hugs the tyre tightly.

4.




Slowly roll forward, making sure that Gecko Traxx is wrapping tightly around the tyre.  
(This takes a little bit of practise!)

5.



Continue to roll your wheelchair forwards providing back tension on Gecko Traxx to ensure a tight fit until the two clips meet.

6.

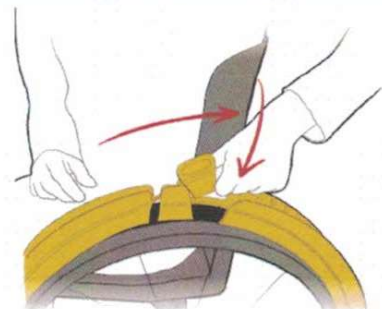


Pull the top clip to provide tension and press it down firmly down until it is flush with the rest of Gecko Traxx.

7.



Repeat steps 1-6 on the 2nd wheel. Alternatively, you can do both wheels at the same time.

8.

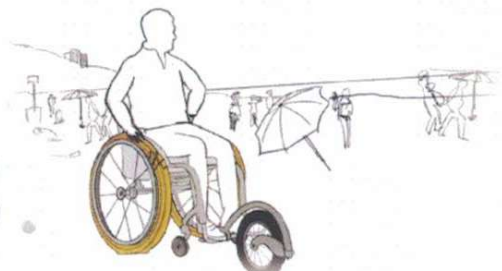

Get out and enjoy the great outdoors!

# Recommendations

1. Never use Gecko Traxx alone. Especially on the beach in case you become stuck. Always have someone available to assist you.
2. Do not take Gecko Traxx into the water with your wheelchair. Water and sand can cause a suction affect causing you to become stuck. Additionally too much salt water contact with your chair can cause the metal to corrode.

# Care Instructions.

1. Wash Gecko Traxx after use and let dry. It is fine to leave them in a mesh bag, just ensure that they dry in a well ventilated area.
2. Ensure that the tyres aren't kinked when stored for long periods of time. This may lead to permanent deformation.
3. Do not store in direct sunlight. This will increase the longevity of Gecko Traxx.

Scan this QR code for a short fitting example and instructional video:





[www.geckotraxx.com](http://www.geckotraxx.com)

1300 988 095